

# Mani-Kare<sup>®</sup> Bandage Dispenser Quick User Guide



**1.** Mark the appropriate screw holes onto the mounting surface.



**2.** Install the appropriate fasteners to the mounting surface.



**3.** To dispense a bandage - firmly grasp the bandage fabric through the wrapper.



**4.** Pull the bottom half of the bandage wrapper and the entire bandage downward.



**5.** A dispensed bandage will be half open and ready to apply to the affected area.



**6.** To install replacement cartridges insert the prongs into the slots through the bottom compartment.



**7.** Properly installed cartridges will lock in place and show through into the top compartment.