



# ABOUT THE MOMCUBE®

MOMCUBE® refrigerators are designed for the storage of breast milk. Units with prefix or suffix “MC” are part of our value line for general purpose storage applications. For more advanced temperature stabilization, we recommend our “MLRS” series.

## Temperature Guidelines for Breast Milk Storage

Storing breast milk in either a refrigerator or freezer will prolong its usability by varying lengths of time.

### In the Freezer

- Freshly expressed will last optimally for up to 6 months, however up to 12 months is acceptable.
- NEVER refreeze human milk after it has been thawed.

### In the Refrigerator

- Freshly expressed milk will last for up to 4 days.
- Thawed milk will last for up to 24 hours.

### Leftovers

- Use within 2 hours after the baby is finished feeding.
- Do not refreeze.

Type of Breast Milk	Room Temperature	Refrigerator	Freezer
	77°F/25°C or colder	40°F/4°C or colder	0°F/-18°C or colder
Freshly Expressed or Pumped Milk	Up to 4 hours	Up to 4 days	Within 6 months is best. Up to 12 months is acceptable
Thawed Milk	1-2 hours	Up to 24 hours	NEVER refreeze human milk after it has been thawed
Leftover from a feeding (baby did not finish bottle)	Use within 2 hours after the baby is finished feeding		

Guidelines courtesy of the Centers for Disease Control and Prevention (CDC) and are intended for healthy full term babies and may vary for premature or sick babies. For more information, contact your health care provider.

In addition to the MOMCUBE® refrigerators and refrigerator-freezers, Accucold offers a line of standalone all-freezers for longer term breast milk storage. We also manufacture and distribute general purpose refrigeration and cooking equipment through our Summit brand, including microwaves, blanket/towel warmers, icemakers/dispensers, ADA compliant appliances, and a large selection of purpose-built refrigerators. If you or your facility need additional storage solutions, we encourage you to visit us online:

More about MOMCUBE®: [momcube.com](http://momcube.com)

Medical & Life Science Equipment: [accucold.com](http://accucold.com)

Residential & Commercial Appliances: [summitappliance.com](http://summitappliance.com)

If you have checked the table above and find that you still need help with your appliance, call our Customer Service facility at **800-932-4267 (Ext. 513)** between 9:00 AM and 5:00 PM ET or visit [www.summitappliance.com/support](http://www.summitappliance.com/support). We will do our best to answer your questions.